



Release Notes –Version 5.4.0

MODALIZER+, 24 January 2019

Summary

Number	Summary
	Improvements
63	Additional DICOM tags parameters in the EXAP XML
	Bugs fixed
61	Viewer stuck after trying to open deleted file from the History
62	MODALIZER+ doesn't keep "don't use camera" setting if there is camera initialization error



Detailed Release Notes

63 - Additional DICOM tags parameters in the EXAP XML

Now file containing data for exchange with EXAP application can be of any format (XML, txt, json etc.).
Default one is XML (PatientDataTemplate.xml) and contains all available fields:

```
<?xml version="1.0" encoding="utf-8" ?>
<PatientData>
  <PatientName>
    $PatientName$
  </PatientName>
  <PatientId>
    $PatientID$
  </PatientId>
  <PatientSex>
    $PatientSex$
  </PatientSex>
  <PatientBirthdate>
    $PatientBirthdate$
  </PatientBirthdate>
  <AccessionNumber>
    $AccessionNumber$
  </AccessionNumber>
  <StudyInstanceUID>
    $StudyInstanceUID$
  </StudyInstanceUID>
  <StudyId>
    $StudyId$
  </StudyId>
  <StudyDate>
    $StudyDate$
  </StudyDate>
  <StudyTime>
    $StudyTime$
  </StudyTime>
  <Modality>
    $Modality$
  </Modality>
  <RequestedProcedureDescription>
    $RequestedProcedureDescription$
```

H.R.Z Software Services LTD
6 David Yellin St., Tel-Aviv 6122101



www.roniza.com
info@roniza.com

```
</RequestedProcedureDescription>  
<ScheduledProcedureStepDescription>  
    $ScheduledProcedureStepDescription$  
</ScheduledProcedureStepDescription>  
<StationName>  
    $StationName$  
</StationName>  
<ReferringPhysiciansName>  
    $ReferringPhysiciansName$  
</ReferringPhysiciansName>  
</PatientData>
```